

Care of your new

# DSG Relaxer™ Splint

**DSG** Dental Services Group™  
The Laboratory Network

[www.dentalservices.net](http://www.dentalservices.net)

This muscle relaxing device has been custom designed for you, personally. It is constructed from heat-molded and heat-cured dental materials and will give you lasting service if proper care is taken.

This device is most often worn during sleep or times of tension and stress. Check with your dental physician to determine your wear-time schedule.

Your relaxer should be removed during eating and rigorous activities, such as sports participation.

You may experience lower front tooth sensitivity to cold temperatures in the beginning. This minimal sensation should not last more than week or two. Also, during the first week or two, you may experience a few minutes of tired jaw muscles in the morning.

Any jaw pain or lasting sensation, not previously noted, should be reported to your dentist immediately.

## Recommended Care

Clean daily with toothpaste and warm water, gently brushing with a toothbrush. Soak weekly for 15 minutes in mouthwash or diluted hydrogen peroxide. Always store in a cool, dry container, well out of reach of children and pets.

Consult your dentist if you have any questions or problem with your new DSG Relaxer.

We share a common mission  
with our customers:

*“To provide the products and services that assist in  
achieving the highest degree of patient satisfaction  
and practice success.”*



**One of Dental Services Group's core values is to help support and promote dentistry.** We do this through our support of Oral Health America and the National Foundation of Dentistry for the Handicapped. In addition to Dental Services Group's corporate support, we raise funds from our suppliers of services and from those interested managers and associates who want to provide a portion of their contributions to advancing dentistry.

