

# How to care for your PermaSoft® Soft Liner



Rinse your denture after every meal. Clean once a day using denture cleanser and an "extra soft" bristle tooth brush.



Soaking your denture up to 10 minutes in a denture cleanser is usually adequate. **DO NOT SOAK "overnight" in cleanser.**



**NEVER SCRUB** the soft liner with a hard bristle denture brush. This will damage the special protective vinyl sealer on the surface of the soft liner.



To prolong the life of your Soft Liner, visit your Dental Professional every six months to reapply vinyl sealer.

**DENTSPLY**  
AUSTENAL

DENTSPLY Austenal, DENTSPLY International Inc., York, PA 17405-0872  
1-800-621-0381 [www.austenal.dentsply.com](http://www.austenal.dentsply.com) [www.dentsply.com](http://www.dentsply.com)  
PermaSoft® is a registered trademark exclusively licensed to  
DENTSPLY International Inc.